## Abscess Remedy Paste

Radiopaque and bactericidal paste for root canal treatment

### Composition

Cresol 9%, Oil of Cinnamon 5.2%, Polyoxymethylene 2.5%, excipient ad 100%.

### Properties

Abscess Remedy Paste is a radiopaque, non-resorbable paste which combines bactericidal and sedative effects. It is used as a temporary root canal filling.

### Indications

- ▶ Treatment of infected 4<sup>th</sup> degree caries, septic pulp necrosis and total pulpal gangrene
- ▶ After pulpectomy, when a permanent root canal filling is to be postponed

### Directions for use

- ▶ After proper cleansing, shaping and irrigation of the root canal, 1 application of the paste kept on place during at least 1 week, should relieve pain and ensure complete disinfection. If required repeat the treatment 1 to 2 times at several days interval
- ▶ Work the paste deeply into the root canal with the aid of a smooth broach or a rotary paste filler. Avoid air pockets or extrusion beyond the apex. A permanent filling can be applied after eliminating of the last applied paste as well as shaping and retreatment of the canal
- ▶ After pulpectomy, obturate the root canal with Abscess Remedy Paste. During the next seat, the paste is to be removed and the mechanical and chemical cleansing of the root canal is to be achieved. Then proceed as per the traditional technique for the permanent obturation of the root canal

### Note

Abscess Remedy Paste is an additional chemical treatment to the usual technique for root canal disinfection. It is not meant to replace this technique.

### Warning

Contains Paraformaldehyde and Formaldehyde. Toxic by inhalation and ingestion. When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

### Storage

Store at room temperature between 15°C and 25°C.

**Shelf life** 3 years.

### Packaging

Jar of 12g

REF

11220

For dental professional use. Made in Switzerland.

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©PD 131014