Abscess Remedy

Radiopaque and bactericidal preparation for root canal disinfection

Composition

Poudre Paraformaldehyde 1.1 %, radiopaque excipient ad 100 %.

Liquide Formaldehyde (40 % vol.) 43.8 %, Beechwood Creosote 9.2 %, Thymol 4.9 %, excipient ad 100 %

Properties

Abscess Remedy is a radiopaque powder and liquid preparation which combines bactericidal and anti-septic effects. It is used as a temporary disinfection treatment of the root canal.

Indications

- ▶ Treatment of infected 4th degree caries, sceptical pulpal necrosis and total pulpal gangrene
- ▶ After pulpectomy, when a permanent root canal filling is to be postponed

Directions for use

- ▶ Pour 3 to 4 drops of liquid onto powder and spatulate into a paste of medium consistency
- ▶ After proper cleansing, shaping and irrigation of the root canal, 2 or 3 applications of the preparation, at an interval of 2 or 3 days, should relieve pain and ensure complete disinfection. Work the paste deeply into the root canal with the aid of a smooth broach or a rotary paste filler. Avoid air pockets or extrusion beyond the apex. A permanent filling can be applied after eliminating of the last applied paste as well as shaping and retreatment of the canal
- ▶ After pulpectomy, obturate the root canal with Abscess Remedy. During the next seat, the preparation is to be removed and the mechanical and chemical cleansing of the root canal is to be achieved. Then proceed as per the traditional technique for the permanent obturation of the root canal

Note

Abscess Remedy liquid can also be used to remove the preparation applied for temporary treatment and also as a general disinfectant for root canals.

Warning

Contains Paraformaldehyde and Formaldehyde. Toxic by inhalation and ingestion. May cause irritation, burns or hypersensitization if in direct contact with the skin. In case of direct contact with the eyes, rinse liberally with running water. In case of ingestion, require the patient to drink a lot of water. Consult a physician.

Storage Store at room temperature between 5°C and 30°C, protected from direct light and humidity.

Shelf life 3 years.

Packaging

Portion 15 g powder + 15 ml liquid

REF
11201

For dental professional use. Made in Switzerland.

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© PD 131014