Alveopaste

Penghwa Iodoform paste – Antiseptic, analgesic and haemostatic alveolar paste

Composition

Benzocaine 26.3%, Iodoform 16.2%, Eugenol 13.6%, Oil of Mint 7.8%, Penghawar-Djambi 3.9%, excipient ad 100%

Properties

Alveopaste combines 4 active substances of long-lasting therapeutic activity. Penghawar-Djambi is a haemostatic, Iodoform is an antiseptic and Benzocaine is a local anaesthetic which acts quickly and remains effective for a long time. Eugenol strengthens the analgesic and antiseptic effects of the other components.

Indications

- ▶ Post-extraction sockets
- ▶ Alveolar osteitis
- ▶ Prevention of post-extraction alveolar complications

Directions for use

As a non-resorbable dressing

- ▶ Apply the paste on a gauze which can be removed after 2 to 6 days or
- ▶ Apply directly into the socket. As the paste is not resorbable, do not suture after application. The paste remains active for several hours. The patient should be advised not to swallow the paste

Warning

In cases where an allergic reaction is noticed, immediately remove the paste already applied and rinse well. When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

Storage

Store at room temperature between 15°C and 25°C, protected from direct light. When closing the jar, make sure the lid is tightly screwed on. Once the jar has been opened, the paste may darken after exposure to air but darkening does not, in any way, alter the effects of the paste.

Shelf life 3 years.

Packaging

Jar of 15g

11490

For dental professional use. Made in Switzerland.

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Composition

Benzocaine 26.3%, Iodoforme 16.2%, Eugénol 13.6%, essence de menthe 7.8%, Penghawar djambi 3.9%, excipient ad 100%

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© PD 140210