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This pack contains powder only

Endofill

Root canal filling material with Dexamethasone

Radiopaque paste for the permanent obturation of the root canal

Composition

Powder Dexamethasone Acetate 0.01 %, Hydrocortisone Acetate 1.0 %, Polyoxymethylene 2.2 %, Thymol iodide 22.5 %, excipient ad 100 %

Liquid Eugenol

Properties

Endofill is a radiopaque preparation for permanent root canal filling. Its composition is well tolerated by tissues and it provides anti-inflammatory, antiseptic and germicidal actions. Before hardening, the paste penetrates the narrowest fissures and maintains its therapeutic effects throughout the treatment until completely set. The final obturation neither retracts nor resorbs.

Indications

Permanent obturation of the root canal either after pulpectomy or after treatment of infected 4th degree cavities, preferably associated with endodontic points.

Directions for use

- > Clean the canal thoroughly and dry with hot air
- > Mix the powder and liquid until to obtain a paste of the required consistency (see list)
- > Work the paste into the root canal with the aid of a probe or a rotary paste filler
Fill the canal completely, avoid air pockets, do not protrude beyond the apex

Average-working times A (21°C/50% humidity) /
Setting times B (37°C/~100% humidity)

	Endofill regular	Endofill slow
Ratio liquid/powder	1/4	1/5
Spatulation	1 min. 20	1 min 20
A	4 h 45	6 h
B (ISO 3107)	2 h 15	5 h 45

Warning

Contains Polyoxymethylene which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns or hypersensitivity. If the product

comes into contact with the eyes, rinse thoroughly with fresh water. In case of ingestion, require the patient to drink a lot of water. Consult a physician.

Storage

Store at room temperature between 5°C and 30°C, protected from direct light and humidity
Shelf-life 3 years in non-opened jar

Packaging	REF regular	REF slow
Portion 15g powder + 15 ml liquid	10270	10275
Bottle of 15 g powder	10271	10276
Bottle of 15 ml liquid	10272	10272

- > FOR DENTAL PROFESSIONAL USE
- > MADE IN SWITZERLAND

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