GB DIRECTIONS FOR USE

Resin Remover

Solution for removing resin-based fillings from the root canal

Composition

N,N-Dimethylformamide 15%, Oil of Citronella 15%, Benzoin Tincture 5%, Tolu Balsam 5%, Essential Oils 4.6%, Excipient ad 100%

Properties

Root canal Resin Remover is a solution to be used specifically for the softening and subsequent mechanical removal of phenoplastic resins from the root canal.

Indication

Removal of phenoplastic resins from the root canal.

Directions for use

(take also notice of the hereafter « warnings »)

- > Clean the access to the root canal
- > Insert a drop of the solution into the pulp chamber
- > Use, successively, a probe and a broach previously dipped in the solution to clear the canal
- > Do not exert undue pressure but repeat the operation as often as necessary
- > Clean the canal, rinse out carefully and dry thoroughly by means of a sterile absorbent paper point
- > Prepare the canal for final obturation

Warnings

- > Resin Remover contains Formamide, a toxic solvent by ingestion or by inhalation and harmful through contact with the skin, the eyes and the soft tissues (R 20/21 and R 36). Avoid, thus, any contact with these tissues and keep the product in the pulp chamber and the root canal, strictly respecting the directions for use
- > Do not keep the product in the canal between 2 sittings
- > Due to the fact that the literature points out a « possible risk during the pregnancy which may cause harm to the unborn child » (R 61), do not use Resin Remover on pregnant women
- > In case of ingestion, significant inhalation or faintness, immediately consult with a physician, informing him of the risks linked to the product
- > Possible allergy to one of the constituents
- > Safety Data Sheet available on our Website or upon request

Storage

- > Store between 5°C and 30°C, away from sunlight
- > Keep away from children, in a safe place

Packaging
Bottle of 15 ml

REF
11556

> FOR DENTAL PROFESSIONAL USE
> MADE IN SWITZERLAND

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