**GB** **DIRECTIONS FOR USE**

## TEMPOTEC

Temporary filling material in paste, Eugenol free

### Composition

Zinc Oxide 46 %, Zinc Sulfate 13 %, Calcium Sulfate 28 %, excipient ad 100 %

### Properties

Tempotec is a slightly radiopaque temporary filling material. Its excellent adhesion to the dentine, sealing qualities and an adequate coefficient of expansion (8 %) ensure a hermetic filling.

The material can be applied to the cavity whether it is either moist or dry.

### Indications

- > Temporary obturation
- > Temporary filling of cavities
- > Temporary setting of root posts

### Directions for use

- > Clean and dry the walls of the cavity
- > Apply Tempotec with a spatula and shape to suit
- > The material begins to dry within 15 minutes after contact with the saliva
- > The final setting is within less than 2 hours
- > Ensure the patient does not exert pressure on the filling for at least 2 hours
- > After treatment, the material can easily be removed with a probe

### Warning and precaution

- > Avoid contact with the gums. May cause irritation when in contact with body tissues other than dentine. May cause hypersensitivity
- > Protect the product from heat or direct exposure to heat sources
- > Do not use the product if you notice any signs of hardening
- > Do not mix the material with any other product

### Product presentation in Jars

Immediately close tightly the original jar after use in order to avoid the absorption of any humidity, resulting in the hardening of the material.

### Product presentation in Blisters

Do not add any liquid or other material into the blister.

### Storage

Tempotec must be stored in well-closed original packaging, with the complete shelter of moisture and a temperature ranging between 15°C and 35°C.

**Shelf-life** 2 years.

### Packaging

Jar of 40 g  
Blister 5 × 0.5 g  
Box 5 blisters of 5 × 0.5 g

REF  
10204  
10205  
10206

- > FOR DENTAL PROFESSIONAL USE
- > MADE IN SWITZERLAND

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